



Get Started Guide




- 1 Remove the battery cover and insert three AAA batteries into the battery holder, observing the polarity.

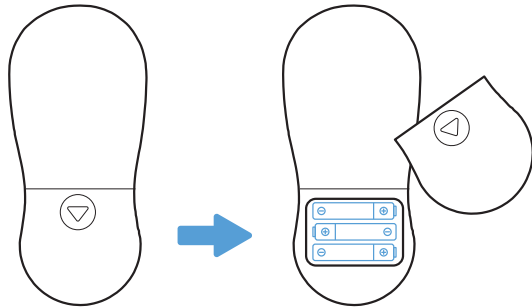


- 2 Cleanse the treatment area of lotion, oil, and grime with a damp cloth or paper towel and dry.

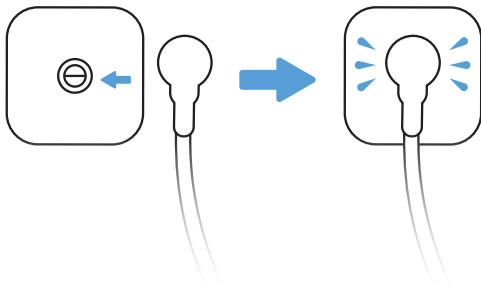


- 3 Attach wire to pads by snapping leads onto pad terminal.

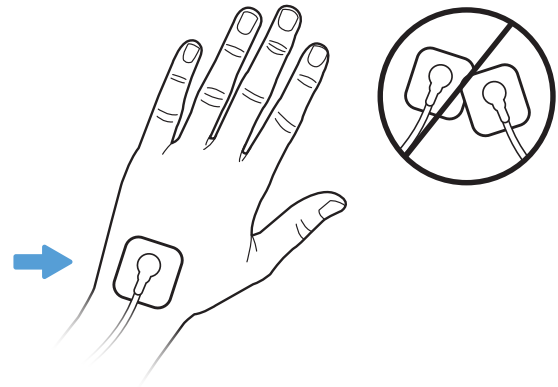

- 4 Place pads over area to be treated. Never allow the pads to touch each other. Refer to manual for suggested pad placements.

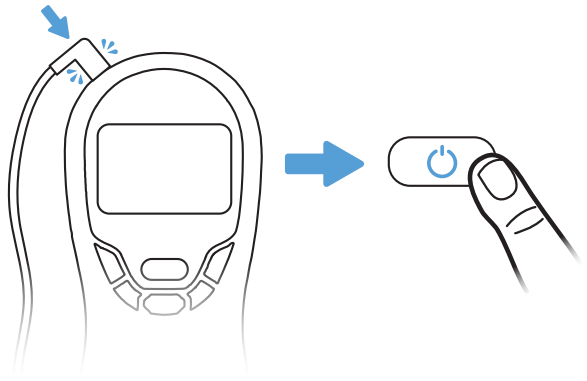

5 Connect wire to device and power on.

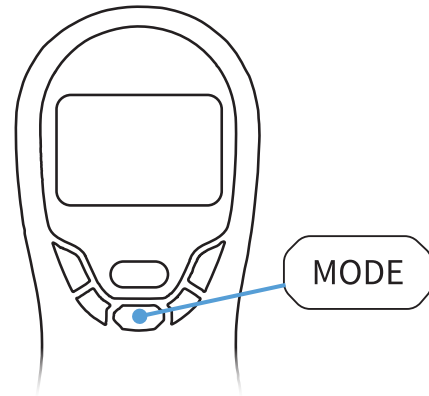

6 Select 1 of 8 modes and begin treatment.



7 The Selection Arrow will blink during treatment cycle indicating the device is working properly. Refer to “Troubleshooting” if the Arrow doesn’t blink.
Note: *Feeling anything during treatment cycles is very rare. The blinking Arrow let’s you know healing is taking place.*



For more information, check out the Owner’s Manual or visit JuvirMedical.com